



CIVCO
MEDICAL SOLUTIONS



Safe. Efficient. Guiding Patient Care.

ULTRA-PRO³

1

attach bracket

Attach custom reusable non-sterile bracket to transducer by aligning locating features. CIVCO manufactures brackets for Aloka, Biosound Esaote, BK Medical, Fukuda, GE Healthcare, Hitachi, Kontron, Medison, Philips, Shimadzu, Siemens, SonoSite, Terason, Toshiba, Ultrasonix and ZONARE.

2

safe. efficient. gu

Six steps for clinical conf

4

place needle guide insert

After selecting the appropriate gauge selector, the insert slides easily into the needle guide.

5

cover up

CIVCO makes it easy to reduce infection rates and cross-contamination with a telescopically-folded CIV-Flex™ cover. Expand your sterile field to 58" by covering the transducer and cord with the sterile latex-free cover.

3



gauge selection

Easy-to-read gauge selector makes it simple to identify and alter gauge sizes in a darkened ultrasound suite. The guide accepts a full-range of gauge sizes including: 11-22GA (19, 21 GA not available).

Guiding patient care.

idence in biopsy procedures.

needle guide assembly

Use proper sterile technique to attach needle guide onto bracket and lock into place with a single click.

6



efficient quick-release

The Ultra-Pro 3 features a large tab for improved quick-release function, allowing easy detachment of the needle from the transducer assembly.

Safe. Efficient. Guiding Patient Care.



Ref. #	Product Description	Qty.
	<u>Needle Guide Kits</u> - Sterile procedure kits.	
610-918	Sterile Ultra-Pro 3™	24
610-901	Sterile Ultra-Pro 3 with 14 x 91.5cm (5.5" x 36") telescopically-folded CIV-Flex™ cover (3D)	24
610-919	Sterile Ultra-Pro 3 with 17.8 x 91.5cm (7" x 36") telescopically-folded CIV-Flex cover (3D)	24



	<u>Transducer Covers</u> - Protective sheath used to prevent cross-contamination.	
610-1000	Sterile 10.2 x 147cm (4" x 58") CIV-Flex cover	24
610-542	Sterile 14 x 91.5cm (5.5" x 36") CIV-Flex cover (3D)	24
610-575	Sterile 14 x 147cm (5.5" x 58") CIV-Flex cover (3D)	24
610-1010	Sterile 17.8 x 147cm (7" x 58") CIV-Flex cover (3D)	24



	<u>System Drapes</u> - Protective sheath used to protect equipment.	
610-1037	Sterile 20 x 28cm (7.9" x 11") polyethylene system drape	24
610-025	Sterile 76 x 76cm (30" x 30") polyethylene system drape	24

CIVCO manufactures brackets for Aloka, Biosound Esaote, BK Medical, Fuduka, GE Healthcare, Hitachi, Kontron, Medison, Philips, Shimadzu, Siemens, SonoSite, Terason, Toshiba, Ultrasonix and ZONARE.

For bracket availability call 319.656.4447 or visit www.civco.com.

Safe. Efficient. Guiding Patient Care.

Introducing the Ultra-Pro 3™, a needle guide that offers all the advantages of needle guidance while maintaining the tactile feel of the freehand technique. The guide features zero needle resistance, offering smooth navigation during puncture procedures.

Accurate Placement

Consisting of a two-part system of a reusable bracket and a disposable snap-on needle guide, the Ultra-Pro 3 provides a safe, reliable and efficient tool for ultrasound-guided procedures including catheter placement, core tissue biopsy, drainage procedures, fine needle aspirations and regional anesthesia. The Ultra-Pro 3 holds the needle as it is advanced, providing a smooth insertion, while allowing you to see what you puncture.



Prevention
Protection
Positioning

- **Prevention:** Transducer and equipment covers protect against cross-contamination providing patient and staff safety.
- **Precision:** Needle guides are designed to reduce technique variability in order to rapidly deliver accuracy and stability for puncture procedures.
- **Positioning:** Needle guides deliver accurate positioning during puncture, providing confidence in hitting the target.

Safe. Efficient. Guiding Patient Care.

ULTRA-PRO³TM

Placing an Order

Contact a CIVCO customer service specialist to place an order or request a product sample.
For international distribution, visit CIVCO Worldwide at WWW.CIVCO.COM

Toll-free within the U.S. and Canada: 800.445.6741

Toll-free Facsimile: 877.329.2482

Direct Dial: 319.656.4447

Facsimile: 319.656.4451

E-mail: info@civco.com

WWW.CIVCO.COM

